




## **marSeal**5 plus

A new dimension in  
intelligent vessel sealing

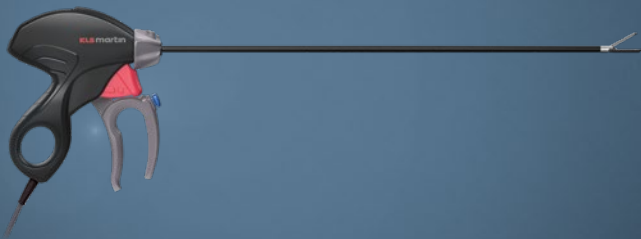



“The new sealing instrument marSeal5 plus is very comfortable to hold. It fulfills all of the ergonomic requirements for a modern sealing instrument.

The fact that the instrument can be reused pays for itself. We value safety, reliability, and cost-effectiveness.”



*Comment from marSeal users in the field of general / visceral surgery and gynecology*



## marSeal5 plus

A new dimension in  
intelligent vessel sealing

Up to 25% faster sealing!

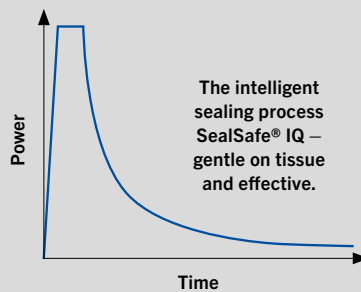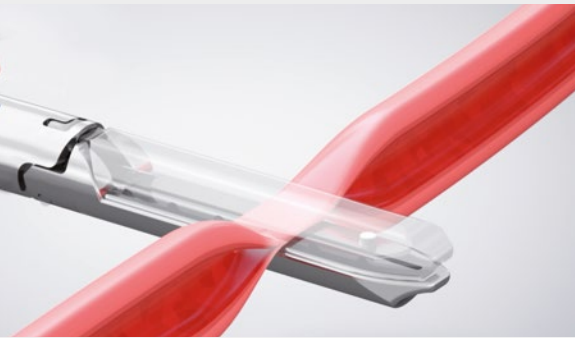

Vessel sealing up to  $\varnothing$  7 mm  
Lateral thermal expansion < 2 mm  
Burst pressure at least 360 mmHg

The combination of the correct current type, SealSafe® IQ, and pressure on the tissue produces a permanent sealing zone. The tissue is cut exactly in the middle.

The impedance-controlled sealing process is continuously monitored. The system only delivers as much power as is required for sealing.

The marSeal5 plus rings in the new generation of bipolar sealing instruments from KLS Martin. It has been systematically refined and tailored to meet the needs of users in terms of ergonomics and the ease with which it can be taken apart and cleaned.

The corresponding current – SealSafe® IQ – is supplied by the high-frequency generator maxium®. Together, they form a bipolar sealing system that converts endogenous structural proteins like collagen and elastin in such a way that veins, arteries and tissue bundles are permanently and reliably sealed. By optimizing the sealing process, sealing time is up to 25% faster.

The marSeal5 plus system features an integrated blade that is used to cut through the tissue in the center of the sealing area immediately after sealing. All this without changing the instrument. And in a single step. This saves valuable surgery time and therefore money.


- ✓ Ergonomic
- ✓ Safe
- ✓ Fast
- ✓ Efficient

**marSeal5** plus  
Ergonomic handle design



**Slim and ergonomic  
handle design**

**Ergonomic  
handle for  
fatigue-free  
working**

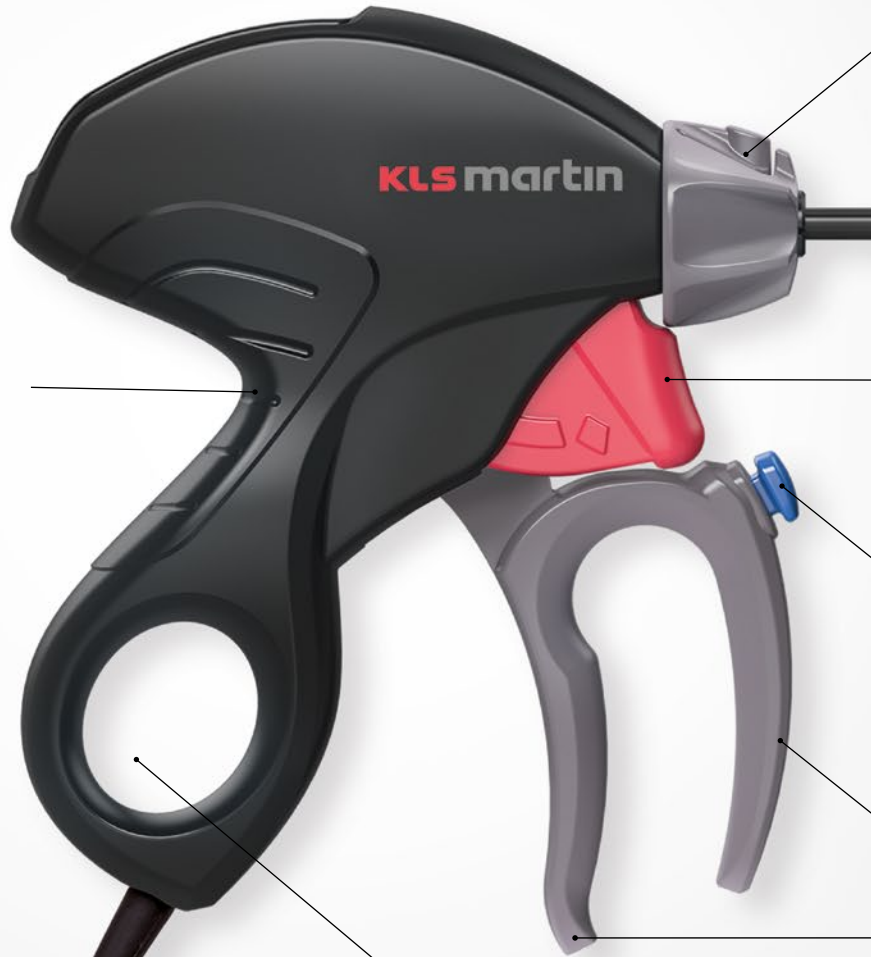



Rotary control  
(rotatable by 360°)

Ø 5 mm

Blade activation  
for tissue cutting

Activation of  
the sealing current



Activation of the sealing current and the tissue separation by means of the index finger – without any additional movement!

Ergonomic grip recesses


Thumb ring for  
overhead working

IQ connector (Plug & Play) for  
automatic instrument identification  
at the maXium® generator\*.



\* maXium® software version 3.423 is needed.


## marSeal5 plus Jaw geometry

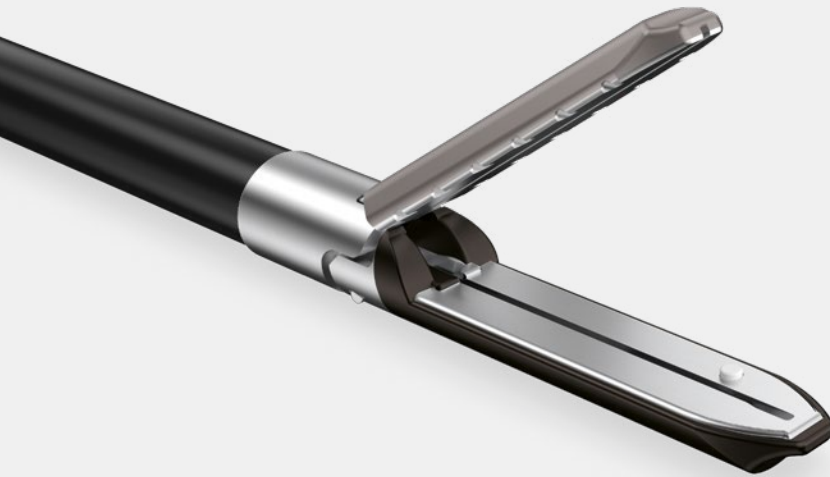

### Special jaw geometry

for preparation and sealing

- Sealing length: 20 mm
- Cutting length: 19 mm

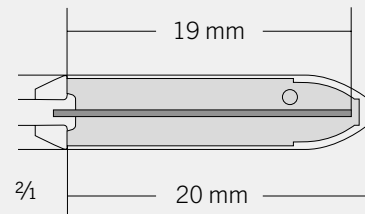

Most of the natural gripping barrel systems have a curved shape.


KLS Martin has transferred this form to the marSeal5 plus. This prevents the tissue which has already been gripped from falling out of the jaws.




The jaws are guided to the targeted tissue.

The jaws close in the front area first. This prevents tissue slippage.

The tissue is compressed by snapping into place. The sealing current can be applied.

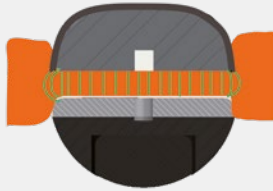


**Through the special coating**  
 on the outside of the jaw part, lateral thermal damage to surrounding tissue is reduced to a minimum.

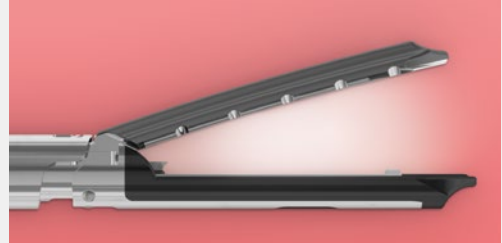

**Bipoling**  
 The instrument can be used with open jaws for surface coagulation. Only the tissue between the jaws is coagulated.

## marSeal5 plus Blade geometry

**The concave blade geometry**  
 ensures that the tissue is automatically guided to the center of the blade. It prevents the tissue from “slipping” on the upper or lower side, so the tissue is not severed.

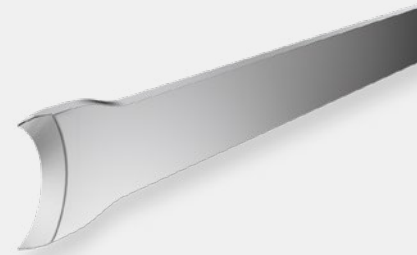

**Tissue separation with marSeal5 plus**



In comparison: sealing instrument with chamfered blade geometry.


Result: The tissue is not severed.

## marSeal5 plus and maXium® – a strong team

The HF generator maXium® supplies the corresponding sealSafe® IQ sealing current. The advantage for you is that when you plug in the IQ sealing instrument, the maXium® unit automatically displays the name of the connected instrument and selects the corresponding current with the relevant default values.


If you do not wish to work with the instrument-specific default values, simply change them as required and save the new values in the usual, program-specific manner. The advantage of this method is that when the IQ instrument is plugged in again, your customized values will appear instead of the default values. This underlines the "Plug & Play" philosophy of KLS Martin.

The marSeal5 plus needs the maXium® software version 3.423.

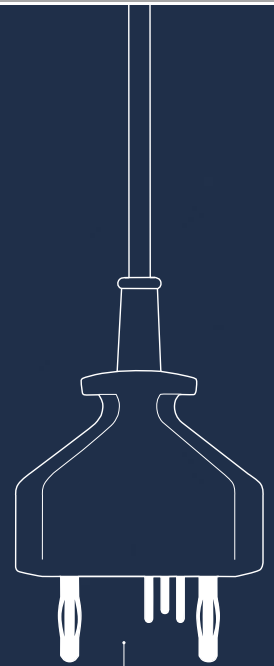

IQ connector for connection to the maXium® generator

Display of the name of the connected instrument

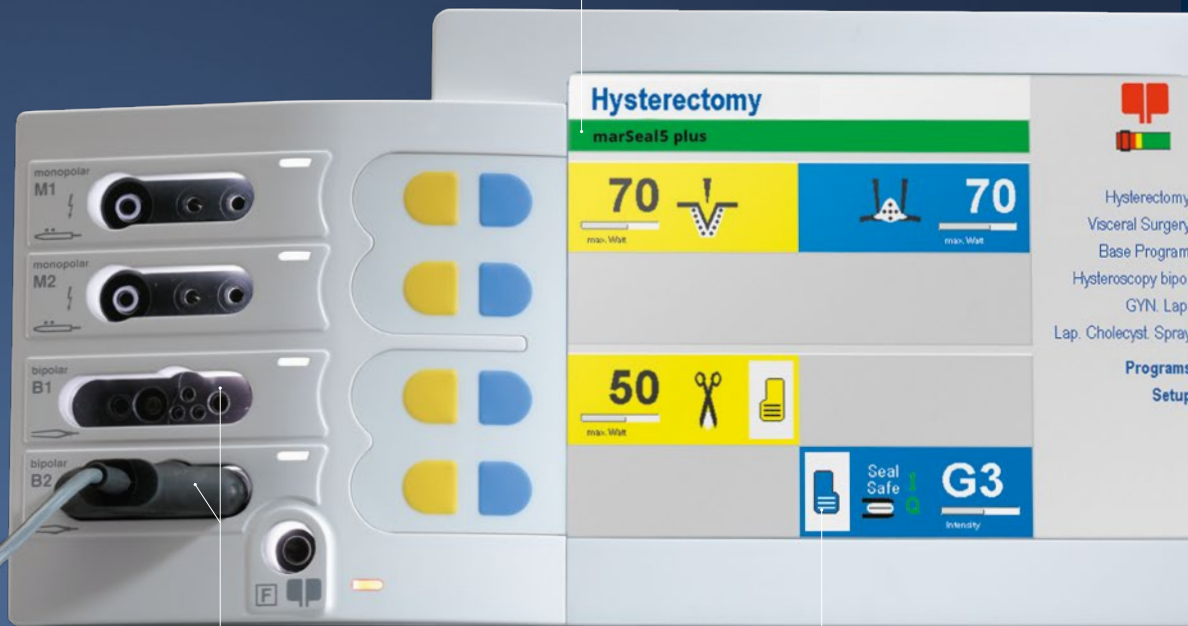


Use of the two bipolar connection sockets

Automatic assignment of the required current mode and set up of the required power output parameters



**marSeal**5 plus –  
a really versatile instrument

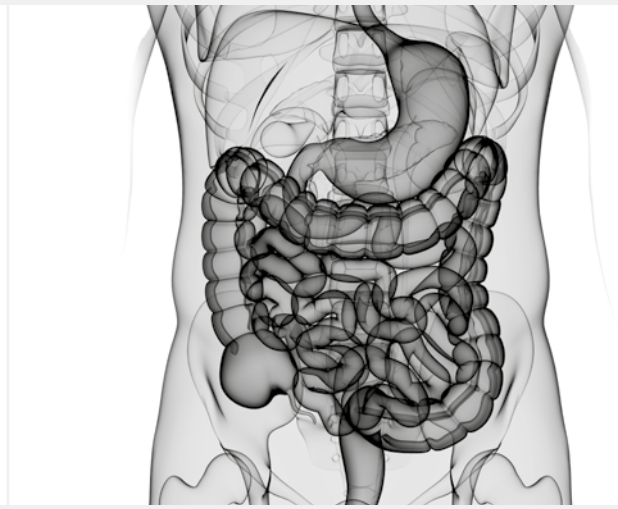

**Applications in gynaecology:**

- Laparoscopic supracervical hysterectomy (LASH)
- Laparoscopically assisted vaginal hysterectomy (LAVH)
- Total laparoscopic hysterectomy (TLH)
- Vaginal hysterectomy
- Abdominal hysterectomy
- Ovariectomy and salpingectomy
- Adhesiolysis
- Cystectomy

**Applications in general and visceral surgery:**

- Colectomy
- Nissen fundoplication
- Splenectomy
- Adrenalectomy
- Nephrectomy

## marSeal5 plus

### Good for the environment and your budget

The disposable proportion of the instrument has been reduced to an absolute minimum. This means that this product is good for both the environment and for your budget.

The instrument features a disposable blade. The remaining components can be easily and quickly cleaned and sterilized using standard processes.

The components of the marSeal5 plus can be dismantled and installed very simply for cleaning:

#### Reusable components



Handle with IQ plug



Outer tube

#### Easy-to-disassemble inner and outer tubes




Inner tube with jaw part

#### Handle is easy to disassemble, due to the innovative magnet system



Open

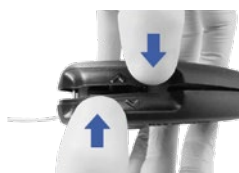

Close

#### Consumables



Blade holder sterile or non-sterile

#### Accessory components for cleaning



Cleaning tool



Rinsing adapter set

## marSeal5 plus

### Cleaning and sterilization tray with lid



The corresponding marSeal5 plus Tray (L x W x H = 532 x 251 x 44 cm) can also be used for cleaning and sterilization. This tray can be ordered optionally under Item Number 55-803-71-04.

## marSeal5 plus

### Item numbers at a glance

Article number	Designation
<b>Complete instrument</b>	
80-633-00-04	marSeal5 plus, complete, consisting of handle, tubular shaft Ø 5 mm, shaft length 370 mm, including cleaning tool and flushing adapter, <b>without blade holder *</b>
<b>Individual components</b>	
80-633-04-04	Handle marSeal5 plus
80-633-05-04	Tubular shaft marSeal5 plus, Ø 5 mm, shaft length 370 mm, consisting of outer and inner shaft, including cleaning tool, <b>without blade holder *</b>
<b>Consumables</b>	
80-633-09-04	Blade holder for marSeal5 plus, 370 mm, sterile, sales unit = 10 pieces
80-633-11-04	Blade holder for marSeal5 plus, 370 mm, non-sterile, sales unit = 10 pieces
<b>Cleaning and reprocessing tools</b>	
80-633-12-04	Cleaning tool for marSeal5 plus
80-633-13-04	Rinsing adapter for marSeal5 plus
55-803-71-04	Cleaning and sterilization tray for marSeal5 plus

\* Blades have to be ordered separately.

## KLS Martin Group

### KLS Martin Australia Pty Ltd.

Sydney · Australia  
Tel. +61 2 9439 5316  
australia@klsmartin.com

### KLS Martin do Brasil Ltda.

São Paulo · Brazil  
Tel. +55 11 3554 2299  
brazil@klsmartin.com

### KLS Martin Medical (Shanghai) International Trading Co. Ltd.

Shanghai · China  
Tel. +86 21 5820 6251  
china@klsmartin.com

### KLS Martin India Pvt Ltd.

Chennai · India  
Tel. +91 44 66 442 300  
india@klsmartin.com

### Martin Italia S.r.l.

Milan · Italy  
Tel. +39 039 605 67 31  
italia@klsmartin.com

### Nippon Martin K.K.

Tokyo · Japan  
Tel. +81 3 3814 1431  
nippon@klsmartin.com

### KLS Martin SE Asia Sdn. Bhd.

Penang · Malaysia  
Tel. +604 505 7838  
malaysia@klsmartin.com

### KLS Martin de México S.A. de C.V.

Mexico City · Mexico  
mexico@klsmartin.com

### Martin Nederland/Marned B.V.

Huizen · The Netherlands  
Tel. +31 35 523 45 38  
nederland@klsmartin.com

### Gebrüder Martin GmbH & Co. KG

Moscow · Russia  
Tel. +7 499 792-76-19  
russia@klsmartin.com

### KLS Martin Taiwan Ltd.

Taipei 106 · Taiwan  
Tel. +886 2 2325 3169  
taiwan@klsmartin.com

### Gebrüder Martin GmbH & Co. KG

Dubai · United Arab Emirates  
Tel. +971 4 454 16 55  
middleeast@klsmartin.com

### KLS Martin UK Ltd.

London · United Kingdom  
Tel. +44 1189 000 570  
uk@klsmartin.com

### KLS Martin LP

Jacksonville · Florida, USA  
Tel. +1 904 641 77 46  
usa@klsmartin.com

Do you know how to get all important information  
about the KLS Martin Energy Devices?

Please download KLS Martin App Energy Devices!  
The App is available for Android and iOS.



[https://itunes.apple.com/de/app/  
klm-martin-energy-devices/id1198171415?l=de&ls=1&mt=8](https://itunes.apple.com/de/app/klm-martin-energy-devices/id1198171415?l=de&ls=1&mt=8)



[https://play.google.com/store/apps/  
details?id=com.klsmartin.energydevices](https://play.google.com/store/apps/details?id=com.klsmartin.energydevices)

### Gebrüder Martin GmbH & Co. KG

#### A company of the KLS Martin Group

KLS Martin Platz 1 · 78532 Tuttlingen · Germany  
P.O. Box 60 · 78501 Tuttlingen · Germany  
Tel. +49 7461 706-0 · Fax +49 7461 706-193  
info@klsmartin.com · www.klsmartin.com

