



Datt Mediproducts Private Limited

# VELNEZ<sup>®</sup>

The Optimal Healing Solution



The painless solution to manage  
surgical procedures

# VELNEZ®

The Optimal Healing Solution



CE  
2460

VelNez® is a biodegradable composite that degrades slowly on application and is intended as an ear dressing. The pain associated with the traditional ear plug removal is completely eliminated since such a procedure is not necessary with VelNez®. It also reduces fibrosis and at the same time promotes healing and reduces the clotting time. During ear procedures, it can be applied to control compression of tissue, middle ear structure and prevents adhesion.

## INTENDED USE

VelNez® is intended to be used as a packing dressing in the middle and outer ear cavity after surgery. It acts as a space occupying dressing, preventing adhesion by separating the compromised surfaces. It also helps in healing and achieving haemostasis.

## INSTRUCTIONS FOR USE

**Step 1: Removal from Package** – Open the blister pack by removing the paper film and take out VelNez®.

**Step 2: Pre-insertion** - If required cut the VelNez® using sterile apparatus.

**Step 3: Insertion** - Using sterile apparatus, place VelNez® inside the ear cavity at the site of application.

**Step 4: Confirm** - Confirm that the device is placed at the intended position. VelNez® will fill itself by absorbing the surrounding fluids and blood.

## CONSTITUENTS

Gelatin (BSE/TSE agent free) Chitosan, Polyvinyl Alcohol, Psyllium Husk.

## WARNING & PRECAUTIONS

- Do not re-sterilize.
- Do not re-use.
- Use the device prior to the “Use by (☞)” specified on the package.
- To be used by Registered Medical Practitioner or a hospital.
- Do not use if package is damaged.
- Discontinue the use if the application area shows signs of infection, irritation, maceration (whitening of surrounding skin) or itching & consult a healthcare professional.

## RISK OF REUSE

VelNez® is supplied sterile for single use only. Do not re-sterilize as this may change product characteristics and lead to failure of intended use of the product. Damaged, unused, opened packages of VelNez® should be discarded as per disposal instructions. Sterility of the product is not guaranteed when the packaging is damaged or unused portion of the device is left in the pack.

## INDICATIONS

As ear pack for hemorrhage control or/and prevention of adhesion between the compromised surfaces after an ear surgery, traumatic bleeding, Tympanoplasty, Tympanostomy or Myringoplasty and other inner and middle ear surgeries.

## CONTRAINDICATION

- Should not be used for the patients with coagulation disorders.

Note: Do not use VelNez® if patient is allergic (Hypersensitive) to it's constituents.

## DISPOSAL OF DEVICE

Discard as per hospital biomedical waste management system meeting local or national regulatory laws.

## PATIENT POPULATION

Patients with age group from 18-60 years of age.

## INTENDED USER

Surgeon, ENT specialist.

## HOW SUPPLIED

Velnez is supplied as a sterile, single-use product in one pack.

## STORAGE

VelNez® should be stored at 4°-30°C. Keep away from direct sunlight. Store in a dry place.

## SHELF LIFE

3 years from the date of Mfg.

## METHOD OF STERILIZATION

Gamma Irradiation.

## AVAILABLE SIZES

SHAPE	SIZE
Square	2cm × 2cm
Rectangular	2cm × 1cm
Cylindrical	20mm × 3mm

## PATENTED

For more information write to Marketing Department



**Datt Mediproducts Pvt. Limited**  
(An ISO 9001 & 13485 Certified Company)

Admin Office: 56, Community Centre, East of Kailash, New Delhi-110065  
Works: 52-54 & 63-64, Roz-ka-Meo Industrial Area,  
Nuh, Distt. Mewat-122 103, Haryana, India  
E-mail: advancedevice@dattmedi.com | Website: www.dattmedi.com  
Customer Care Toll Free No.: 1800-1030-707

## ECREP

mdi Europa GmbH Langenhagener Str. 71  
D-30855 Hannover-Langenhagen, Germany Phone: +49-511-3908 9530  
Internet: www.mdi-europa.com Trade Register Hanover HRB 58423